PureWhite

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VALIDEZ: 24 meses.

Pure White is a professional dental whitening system containing 35% hydrogen peroxide, indicated for the whitening of vital and non-vital teeth. This product is intended for in-office use only under the supervision of a licensed dental professional. It must always be used with an effective gingival isolation system such as Gingi Dam – Light-Cured Gingival Barrier to protect soft tissues. After thorough mixing of both phases, a hydrogen peroxide-based gel is formed with a neutral pH (6.0 to 7.0). The neutral pH minimizes effects on dental structures. Additionally, Pure White contains desensitizing agents in its formula to enhance patient comfort during treatment.

INDICATIONS FOR USE

Indicated for the treatment of chronic extrinsic discoloration of vital and non-vital teeth, which cannot be removed by conventional cleaning procedures. May be used in conjunction with blue light sources to assist in the acceleration of the whitening process.

DIRECTIONS FOR USE

Clean the teeth thoroughly and record the initial shade using a VITA shade guide or clinical photographs.
Dry the teeth completely and perform careful relative isolation using Gingi Dam – Light-Cured Gingival Barrier, carefully outlining the gingival tissues. Light cure each tooth for 10 seconds.
Connect the two syringes from the Pure White kit. Once securely attached, mix by alternately pushing the plungers at least 10 times or until a homogeneous gel is formed.

Self-Mixing System: The self-mixing system simplifies the combination of the peroxide with the thickening agent. The two syringes are connected by a system that allows the material to pass back and forth between them, effectively blending both phases. After mixing, transfer the gel into a single syringe for application.

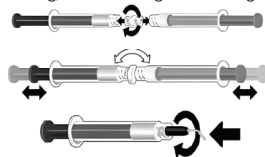
Figure 01 – Illustration of the Self-Mixing System.

Once the gel is prepared, remove the connector and attach the applicator tip to the syringe.

GEL APPLICATION PROTOCOL

- Apply the mixed whitening gel to the entire buccal and proximal surfaces of each tooth to be whitened. The gel layer should be approximately 1 mm thick.
- Leave the gel in contact with the teeth for up to 45 minutes. If oxygen bubbles appear, release them using a disposable brush.
- If the patient reports sensitivity, remove the product immediately and assess the cause (e.g., cracks, microleakage, exposed dentin).
- After 45 minutes, suction the gel using an appropriate aspirator tip and rinse thoroughly with water. Remove the Gingi Dam using an explorer, carefully detaching it from the tissue. Apply Sensis 2% Desensitizing Gel for 10 minutes.

PRECAUTIONS AND CONTRAINDICATIONS

Conduct a detailed patient anamnesis before treatment;
Use only in dental offices with appropriate personal protective equipment (PPE) including gloves and safety goggles;
Avoid contact of the gel with soft tissues and gingiva;
Not recommended for patients who have recently undergone gingival surgery, present gingival inflammation, or are allergic to any of the product's components;
Contraindicated during pregnancy, breastfeeding, and in patients under 15 years of age;
For professional use only by licensed dental personnel;
This product is non-sterile;
Read instructions carefully before use;
Do not ingest. Avoid contact with skin, oral soft tissues, and eyes;
Patients are advised to wear protective eyewear during treatment.

ADVERSE EFFECTS

Ingestion of large quantities may cause nausea; Inform patients about the possibility of temporary tooth sensitivity, which is common during whitening procedures. If sensitivity persists, treat affected teeth with Sensis 2% – Desensitizing Gel, which is indicated for moderate to high sensitivity. Discontinue use if any allergic reaction occurs.

COMPOSITION

Pure White Whitening Gel: Hydrogen peroxide, stabilizer, neutralizing agent, thickener, desensitizing agents, structuring agent, preservatives, blue pigment, enzyme, and deionized water.
Gingi Dam – Gingival Barrier: HEMA, UDMA, photoinitiators, inert filler, and pigment.

STORAGE CONDITIONS

Keep the product in its original, tightly sealed packaging;
Protect from direct sunlight;
Store between 5°C and 28°C (41°F–82°F);
Do not freeze or expose to excessive heat.

Contents:

1 syringe of Hydrogen Peroxide with 1.2g
1 syringe of Thickener with 0.4g
1 syringe with Gingi Dam (gingival barrier) 1g
1 syringe connector

Shelf Life: 24 months.

  +55 (47) 98882-8351
fabrica@villevie.com www.villevie.com
Fabricante:
DENTALVILLE DO BRASIL LDA - CNPJ: 01.101.363/0001-86
Rua Adv. Miguel Ceral, 645 - Nova Brasília-Joinville-SC 89214-610
Resp. Técnico: Aldo Castellani - CRO-SC 13.400.397 13°R
Reg. ANVISA: 10344119007
Tel.: +55 (47) 3454-2200
Hecho en Brasil

  +55 (47) 98882-8351
fabrica@villevie.com www.villevie.com
Manufacturer:
DENTALVILLE DO BRASIL LDA - CNPJ: 01.101.363/0001-86
Rua Adv. Miguel Ceral, 645 - Nova Brasília-Joinville-SC 89214-610
Resp. Técnico: Aldo Castellani - CRO-SC 13.400.397 13°R
Reg. ANVISA: 10344119007
Phone: +55 (47) 3454-2200
Made in Brazil