PT INDICAÇÃO

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VALIDEZ: 24 meses

EN INDICATION

Indicated for bracket bonding to the dental surface (enamel) for orthodontic treatments.

DIRECTIONS FOR USE

- 1) Prepare the dental surface for bonding by isolating the area, cleaning, and drying thoroughly;
- 2) Apply phosphoric acid etchant to the area where the bracket will be bonded and let it act for at least 30 seconds;
- 3) Rinse the surface with water and dry completely (ensure no moisture remains);
- 4) Apply orthodontic cement to the base of the bracket;
- 5) Position the bracket on the tooth surface and apply firm pressure (the thinner the layer between tooth and bracket, the better the adhesion);
- 6) Remove excess cement;
- 7) Light cure the cement for at least 40 seconds.

PRECAUTIONS

Avoid applying the product on the patient's gingiva if they have recently undergone any oral surgical procedure or present any gingival irritation. Incomplete curing or excessive exposure time may lead to mild gingival irritation in some cases.

STORAGE CONDITIONS

Keep the product in its original closed packaging; Protect from direct sunlight; Store at temperatures between 41°F–82°F.

COMPOSITION

Methacrylate monomers, nanometric silica, sodium fluoride, aluminum boro-silicate glass, photoinitiator, and co-initiator.

Shelf Life: 24 months.

Fabricante / Manufacturer:

DENTALVILLE DO BRASIL LDA - CNPJ: 01.101.363/0001-86 - Rua Adv. Miguel Cercal, 645 - Nova Brasília-Joinville-SC 89214-610 Resp. Técnico: Aldo Castellani - CRQ-SC 13.400.397 13°C
Reg. ANVISA: 10344119004 - Fone: +55 (47) 3454-2200
Indústria Brasileira - Hecho en Brasil - Made in Brazil