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EN

DENTAL BLEACHING PRODUCT

MIX NIGHT 16% is a dental bleaching gel based on 16% carbamide peroxide indicated for at-home vital tooth bleaching.

PRODUCT INDICATIONS

It is indicated for at-home vital tooth bleaching. Use only under dentist's supervision.

PROFESSIONAL INSTRUCTIONS

MIX NIGHT 16% should be used in a custom-fitted tray for upper and/or lower arches, for 3 to 4 hours daily. The treatment should last 10 to 14 days, according to the patients needs, evaluated and supervised by the professional. The product can be used overnight or at daytime, according to the professional instructions. The results are visible soon after the first days of use.

MIX NIGHT 16%: The 16% concentration is more suitable for severe tooth discoloration, more challenging cases and when a faster bleaching is desired. Before starting treatment, evaluate the general patient's health as well as the gingival health. Dentin hypersensitivity, caries lesions, dental fractures among other abnormalities should be treated before the beginning of the bleaching protocol. The case must be followed up for evaluation and to guide the patients at least every 7 days. Color Registration: Register the individual color of each tooth to be bleached. As a suggestion take some photographs from the patient's arches at the beginning and at the end of the treatment for better visualization of the results. Instruct the patient to

place the bleaching gel into the custom-fitted tray as instructions for use (for the patient). Some patients may experience severe tooth sensitivity during the bleaching treatment. In these cases, the desensitizing SENSIS 0.2% or 2% gel should be applied. If sensitivity persists or if an allergic reaction to the product occurs stop the bleaching treatment immediately. Inform patients about the level of whitening to be achieved for each individual case as well as treatment time. These factors are dependent on the severity of the discoloration. The treatment may highlight some hypo-calcification areas during dental bleaching; but they tend to disappear with the rehydration of the tooth or after local fluoride application. In case there is need to replace restorations or dental prosthesis, it is recommended to delay this procedure for one week after completion of treatment. This time delay is required to allow for color stabilization and also to prevent bonding failure when restorative materials are bonded to the bleached teeth.

INSTRUCTIONS FOR USE (FOR THE PATIENT):

1. Place a small drop of the gel in into every compartment of the bleaching tray for the teeth undergoing treatment.
2. Seat the tray on the dental arch with a slight pressure with the aim to distribute gel around all dental surfaces
3. Wipe away excess gel with the fingers or a gauze.
4. Leave the gel in contact with the teeth surfaces for 3 to 4 hours daily).
5. After that, remove the tray from the mouth and rinse the teeth throughout. Do not swallow the gel.
6. Rinse the bleaching trays with water and brush it with a toothbrush before storage.
7. Do not brush your teeth with toothpaste before product

use. This may cause tooth sensitivity. After product use, wait for 30 minutes to brush your teeth.

IT IS IMPORTANT THE PROFESSIONAL INSTRUCT THE PATIENT

Restorative materials and dental prosthesis do not bleach with product use and should be evaluated in the end of the bleaching treatment. Teeth darkened by antibiotics (tetracycline-stained teeth), non-vital tooth or teeth with a grayish color do not respond as well as sound teeth to vital tooth bleaching. Avoid eating acidic foods or beverages during the treatment as this may increase the tooth sensitivity likelihood. Also avoid colored foods and drinks as it may jeopardize the treatment outcomes.

PRECAUTIONS AND CONTRAINDICATIONS

Do not swallow the gel. Do not use the product after the expiration date. Use the product only under professional's supervision. In case of contact with eyes, rinse throughout with water for 15 minutes and seek for medical assistance. The product is contraindicated during pregnancy and lactation, for patients allergic to components of the formula, with periodontal disease or stomach problems and patients that had recently undergone periodontal surgery.

SIDE EFFECTS

During vital tooth bleaching, tooth sensitivity may occur depending on the patient. This sensitivity is transient. In case of severe tooth sensitivity, apply the desensitizing gel SENSIS 0.2% or 2%. If tooth sensitivity persists or any allergic reaction occurs, interrupt the treatment immediately.

STORAGE AND TRANSPORT

Store the product in temperatures below 25°C in a dry and cool place protected from direct light. Observe the expiration date before use. Do not use if the package is violated. In case you notice any change in the product or violation of the package, contact Villevie to be instructed about the procedures to be followed. Do not expose to excessive heat. Prolonged exposure to temperatures above 40°C can alter the chemical composition of the product. There are no additional recommendations regarding transportation.

COMPOSITION

Carbamide peroxide 16%, purified water, propylenoglycol, sodium fluoride, potassium nitrate, tutti-frutti flavor, thickener, preservative, neutralizer and stabilizer.

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